



# A breakthrough in waterless humidification

Introducing HumidX for AirMini, ResMed's first waterless humidification system that's been designed to deliver a comfortable therapy experience for your patients at home, or while they're away.




### Technology that serves you and your patients

The HumidX<sup>\*</sup> is inserted in front of the vent of the patient's mask, allowing it to capture the air's warmth and humidity before it escapes through the vent.<sup>\*</sup>

It is ResMed's ground-breaking ActiveAir<sup>™</sup> technology that makes waterless humidification in AirMini possible. This technology allows the vent to be built directly into the AirMini mask system and located downstream of the HumidX without compromising on comfort or seal.

HumidX technology is currently available for AirFit™ P10 for AirMini nasal pillows mask, AirFit N20, AirFit N20 Classic, AirTouch™ N20 and AirFit N30 for AirMini nasal masks, and AirFit F20 and AirTouch F20 full face masks.

#### ActiveAir vent system

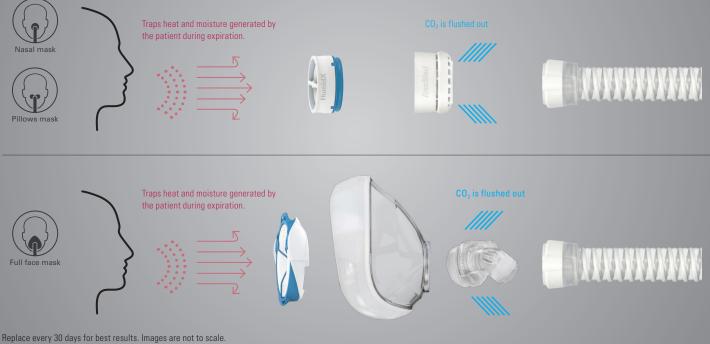

#### How it all works

As patients breathe, heat and moisture from every exhale is captured and stored within the HME's tiny paper ridges.

When they inhale, this heat and moisture is released to provide warm and comfortable air throughout the night.

## Technology reinvented

Some hospital devices have been using waterless humidification for over 50 years.

What sets ResMed's HumidX apart from other CPAP waterless humidification solutions is that it passes air through an exchange before it reaches the mask's vent. In doing so, heat and moisture loss is minimised.



HumidX for nasal and nasal pillows masks ResMed's standard waterless humidification system designed for most climates and sleep environments.



ResMed Ltd

HumidX Plus for nasal and nasal pillows masks Optimised for dry climates and high altitude environments where humidity levels are very low, e.g., aircraft.

#### HumidX F20 for full face masks<sup>+</sup> ResMed's standard waterless humidification system designed for most climates and sleep environments.

#### Product codes

For nasal and nasal pillows masks		
HumidX 3PK	38809	
HumidX 6PK	38810	
HumidX Plus 3PK	38812	
HumidX Plus 6PK	38813	

_		-	
For	full	face	masks

HumidX F20 3PK	38012
HumidX F20 6PK	38013

# Learn more about HumidX at ResMed.com/HumidX

